(ESP)

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CLIP-FAB®

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INSTRUCTIONS FOR USE

CLIP-FAB®

Autonomous disposable device for endotracheal tube fixation

Each **CLIP-FAB** device is supplied in a sealed envelope and conveniently labelled.

Warnings.

A Warning states a condition, hazard, or unsafe practice that can result in serious personal injury or death.

- This device should only be used by trained personnel in accordance with the Indications and Setup instructions.
- Prior to use, inspect all parts to ensure they are free of damage and working properly.
- The patient should be constantly monitored while this product is in use.
- Designed for the fixation of endotracheal tubes or double lumen tubes, with internal diameters between 7 and 9 mm.
- Other support methods will be used if the geometry or physical characteristics of the patient require it.

Indications for use

The Clip-Fab disposable device is designed for endotracheal tube fixation, reducing the risk of accidental extubation.

Setup instructions

Once the patient is intubated, and muscularly relaxed, insert the balloon of the device inflated to medium pressure (3-5cc) after the tooth, and face the inner clamp in the direction of the tube.

Apply only the pressure necessary to securely fix it inside.

Proceed to regulate the inflation of the balloon to fully adapt to the patient's intraoral profile (5-7cc).

Patient can be extubated without removing the device.

Cautions

A Caution states a condition, hazard, or unsafe practice that can result in minor personal injury or damage to the device.

- Before use, check that the balloon and valve of the device are working correctly.
- The device should only be attached to the patient after insertion of an advanced airway device.
- This device is not intended for use on pediatric patients.
- This device is not intended for use on toothless patients.
- This device is for one use only. Not reusable.
- Do not use this product if its packaging is damaged or in poor condition.

Contraindications and side effects

The incompatibilities or side effects were not detected in the nature of the device and in the tests performed.

Maximum time of use.

Class I disposable sanitary device for short-term use of up to 30 days in the oral cavity.

Manufacturer's recommendation: replacement every 24 hours.

Materials

Part	Material
Bodie and clamp	Polycarbonate
Disc	PVC
Hi-Lo Balloon	PVC
Inflation Valve	PVC

This device was not made with natural rubber latex.

Dispose of according to local protocol.

Note

The syringe for inflation of the balloon is not provided with this device.

Ref: 011

Clip-Fab® is trademarks owned by
Clip-Fab R&D Medical Instruments S.L.
Ownership and all rights reserved.

Spanish Health License Number 7454-PS
Agencia Española de Medicamentos
y Productos Sanitarios
Medical Device Regulation

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